



**Food & Inhalant
Antibody Assessment Panels
Candida Antibodies & Antigen
Celiac Antibody Profile
Painkiller Antibody Panel**

*Celiac & Specific-IgA panels not suitable for breast-fed infants.
All other panels not suitable for ages less than 6 months.*

Collection Kit Contents
· 2 10-ml Red Top Vacutainer
· 1 10-ml Serum Transfer Tube
· 1 Disposable Transfer Pipette
· 1 Ziploc bag · 1 Absorbent Pad
· 1 Polyfoam Packer · 1 Mailing Box
· 1 Test Requisition Form
· 1 FedEx Clinical Pak & Billable Stamp

Diet & Medications

1. It is advised to adhere to usual dietary habits consuming a variety of foods when possible. Avoid any food that may have potentially resulted in a previous adverse reaction or anaphylaxis. Secondary exposure may prove fatal. For **Celiac Panel**, individuals on a gluten-free diet prior to testing may show low or falsely negative serological values.

2. Immunosuppressive drugs such as oral or intranasal corticosteroids (prednisone, beclomethasone, fluticasone, triamcinolone), and topical cortisone suspensions and creams, may affect test results for antibody assessment. The suggested time period to abstain from these is 4 weeks.

13500 Linden Ave North
Seattle WA 98133 USA

Phone: 877.318.8728
Phone: 206.365.1256 (Seattle)
Fax: 206.363.8790
Email: cservice@usbiotek.com

To order supplies call US BioTek Laboratories during our normal business hours: 8:30am to 5:00pm PST.

11.03.09

IMPORTANT SHIPPING INSTRUCTIONS

- Clearly print Patient's Name & Sample Collection Date on **Serum Transfer Tube** and **Test Requisition Form**.
 - Test Requisition Form** must also include:
 - Patient's Date of Birth ·Practitioner's Signature ·Date Sent
 - Test Request ·Billing Information ·ICD-9 code (if being billed through insurance).
 - US BioTek is NOT a Medicare or Medicaid provider.**
 - Our test procedures are not reimbursable by both.**
 - Missing or incomplete test request information will delay processing and extend turn-around time.**
 - Complete **FedEx Express Billable Stamp** and stick onto **Clinical Pak**.
 - Stamps are pre-paid for 2nd Day Delivery for lower 48 states. Contact FedEx for pick up: 1.800.463.3339.*
 - Do not schedule for a weekend pick-up. Client will be responsible for weekend shipping rates.
 - Alaska, Hawaii and Canada clients: Pre-paid shipping does not apply. Client is responsible for the shipping cost.
 - For tracking purposes keep a copy of your **Shipment No.**
 - Specimen may be safely shipped at room temperature.
- Food, Inhalant & Candida Panels - The lab must receive specimen within 14 days at room temperature from date of collection.
- Celiac Antibody Panel -The lab must receive specimen within 21 days at room temperature from date of collection.
- Painkiller Antibody Panel - The lab must receive specimen within 7 days at room temperature from date of collection.

SPECIMEN COLLECTION INSTRUCTIONS

Serum Requirements:

- Serum volumes are listed on the Test Requisition Form and in our Service Directory. An insufficient sample may result in the inability to complete testing for all panels ordered. Notification of the panels that could not be completed is issued with the partial report. Follow collection instructions carefully. There is a \$15 fee for submittal of an unacceptable specimen.
1. Draw the blood utilizing the **Red Top Vacutainer** tubes provided. 2 fully drawn 10 ml sized Red Top OR SST tubes will usually provide about 6-8 ml of serum. If 2 Red Top OR SST tubes are not sufficient to provide the volume required for the panels ordered, a third tube may be needed.
 2. Allow the blood to clot at room temperature for 30 minutes.
 3. Centrifuge at 3000 RPM for 15 minutes.
 4. With the **Disposable Transfer Pipette**, transfer serum from the **Red Top Vacutainers** into **Serum Transfer Tube**. Do not transfer red blood cells to the **Serum Transfer Tube**.
 5. Secure tightly.
 6. Wrap **Serum Transfer Tube** with **Absorbent Pad** and place it into the **Ziploc Bag**. Seal contents into **Polyfoam Packer** and then into the **Mailing Box**. Include **Test Requisition Form** inside the **Mailing Box**.
 7. Put the **Mailing Box** into the **FedEx Express Clinical Pak**. Seal securely.